

PREVENTIVE MAINTENANCE CHECKLIST SIX (6) MONTH

SERIAL NO. _____ DATE _____

MACHINE I.D. NO. _____ MACHINE HOURS _____

TECHNICIAN(S) _____ LABOR HOURS _____

See the Referenced Section of the 2008T Preventive Maintenance Procedures (part number 508033 Rev. O) for detailed instructions to perform the following:

<u>Reference Section</u>	<u>Procedure</u>	<u>Value</u>	<u>Completed By:</u>
2.1	Clean Filters.....(Replace If Necessary)	_____	_____
2.2	Inspect Pre-UF Pump filter.....(Replace If Necessary)	_____	_____
2.3	Inspect UF Pump Check Valves.....(Replace If Necessary)	_____	_____
2.4	Check High Voltage AC Connections.....(<0.2 ohms)	_____	_____
2.5	Calibrate UF Pump.....(Range: 24 ± 0.1cc at 24 UF Pump strokes)	_____	_____
2.6	Check Conductivity Calibration.....(Range: Δ ± 0.1mS/cm)	Machine Display _____ External Meter _____	_____
		(Range: Δ ± 0.2mS/cm) _____ TCD _____ External Meter _____	_____
2.7	Check Temperature Calibration.....(Range: 37 ± 0.3°C)	_____	_____
		(Range: 39 ± 0.3°C) _____	_____
2.8	Check Alarm Operation and Pressure Holding Test (N/A for 2008T BlueStar Premium)	_____	_____
2.9	Verify pH (N/A for 2008T BlueStar Premium).....(Range 6.9 to 7.6)	_____	_____
2.10	Check Power Failure Alarm (N/A for 2008T BlueStar Premium).....	_____	_____
2.11	Final Checks.....	_____	_____

NOTES:
